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## Choice of pillars

Equator pillars (Rhein'83, Bologna, Italy) were chosen for their reduced height and their horizontal dimension inside the prosthesis. Furthermore, the degree of parallelism between the pillars could be widely managed by this type of retention. The pillars were positioned in the mouth with a Torque of 25 N (Fig. 54).

## Preparation for finalization of the prosthesis

Before finishing the prosthesis, it was considered necessary to relin the plates used for the set-up so as to compensate for the tissue modifications that occurred after surgery and during the healing period. When recording the mandibular impression, it was also decided to record the position of the Equator pillars (Figs. 55 and 56).

## Space evaluation

The models obtained from relining the plaques were mounted in the articulator (Stratos 200, Ivoclar Vivadent, Schaan, Liechtenstein) (Fig. 57). This allowed an evaluation of the space available for positioning the retentive components in order to decide if a reinforcement structure needed to be constructed.

In this case, it was decided that the space available was more than enough to exclude the construction (Figs. 58 and 59).

It was therefore decided to finalize the prosthesis and proceed with the uncovering of the implants after delivery, then position the pillars, and finally the direct fixing of the retentive components in the prosthesis.

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